LIMISAN PRO

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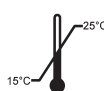
medical adhesive remover

TO REMOVE MEDICAL ADHESIVES

READ THIS LEAFLET CAREFULLY BEFORE USING LIMISAN. KEEP THIS LEAFLET, YOU MAY NEED IT AGAIN.

What is Limisan?

Limisan is a painless remover of several adhesives. Limisan is designed to facilitate easy, pain free and non-traumatic removal of all types of adhesives, including plasters, tapes, dressings and patches. This is friendlier for the skin and can help promote better wound healing as it reduces the damage caused to the skin and wound area by removing plasters and dressings. Limisan is a spray which ensures easy application and it will work within seconds. The spray can be used upside down. Limisan is a very mild product, it will not sting! Limisan will dry very quickly and will therefore not influence the adhesion of the next plaster. Limisan consists of:

1. A 50 ml spray bottle, containing the following ingredients: Silicons, Bisabolol, Spearmint, ARC-extract, Butane, Propane;
2. This information leaflet containing all the information you need to know about Limisan and how to use Limisan. Keep this leaflet for your reference.

How does Limisan work?

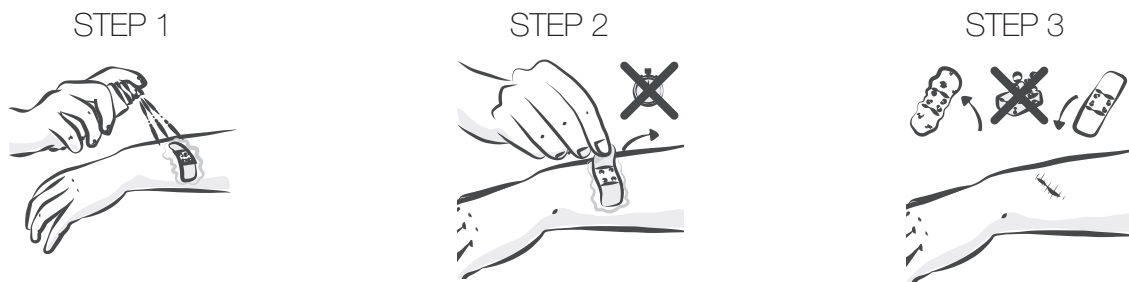
Limisan contains a blend of ingredients that quickly form a layer between the adhesive and the skin. This layer interrupts the adhesive bond, causing a total loss of adhesion and allowing the plaster to be gently lifted away. All the ingredients in Limisan evaporate quickly leaving no residue, enabling a new adhesive to be applied quickly.

How to use Limisan on plasters or other adhesives?

STEP 1: As soon as you want to get rid of your plaster or other adhesives from the skin, you can spray Limisan around the edge and on top of the adhesive until it looks wet. Avoid getting the product into the eyes or other mucous membranes.

STEP 2: No waiting required. Directly peel the adhesive away from the skin. Spray additional Limisan to ease removal if necessary. Remove the adhesive quickly to avoid that the adhesive residues will stay on the skin. Do you still have any adhesive residues left? You can easily remove them by spraying Limisan on a tissue and remove the residue with this tissue.

STEP 3: Limisan will dry very quickly and it is not needed to clean the area.

How to use Limisan on adhesive residue?

STEP 1: As soon as you want to get rid of any adhesive residue from the skin, you can spray Limisan on a tissue.

STEP 2: Rub the wet tissue on the residue and the residue will be peeled off the skin.

How to use Limisan around the eyes?

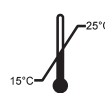
STEP 1: As soon as you want to get rid of your adhesive around the eyes spray Limisan in the cap of the bottle.

STEP 2: Dip a cotton swab in the liquid in the cap.

STEP 3: Apply the wet cotton swab on the adhesive until it looks wet and gently peel the adhesive away from the skin. Use additional Limisan to ease removal if necessary.

Danger!

- Limisan should not be used if the person is allergic to any of the ingredients contained in Limisan;
- Even though Limisan is proven to be "very good" during a skin safety test, we recommend to test Limisan on a small skin area prior to first use;
- Extremely flammable aerosol. Pressurised container: May burst if heated;
- Avoid getting the product into the eyes or other mucous membranes. In case of contact with eyes, rinse thoroughly with water. Do not swallow. If swallowed, do not induce vomiting, seek medical attention;
- Do not inhale. Use only in well ventilated areas. If inhaled, remove person to fresh air and keep at rest in a position comfortable for breathing;
- Discontinue use in case of any undesirable effect and consult your doctor. If medical advice is needed, have product or label at hand;
- Store at room temperature and do not expose to temperatures above 50 °C/122 °F. Store in a well-ventilated place. Protect from sunlight and do not pierce or burn even after use;
- Keep out of reach of children;
- Keep away from heat, hot surfaces sparks, open flames and other ignition sources. No smoking. Do not spray on an open flame or other ignition source. Do not pierce or burn, even after use;
- For external use only;
- Do not use after the best before date;
- Avoid release to the environment. Very toxic to aquatic life. Toxic to aquatic life with long lasting effects. Collect spillage. Dispose of contents/container in accordance with local/ regional/ national/international regulations;
- Any serious incident that has occurred in relation to the product should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

Fabrikant / Manufacturer:
Archy Medical B.V.
Westhove 119
1187 DB Amstelveen
The Netherlands

Distributeur / Distributor:
Archy Medical B.V.
Westhove 119
1187 DB Amstelveen
The Netherlands

LSP16NL102
59-31375-0131
www.limisan.com
Revised: February 2021